



activheal

WOUND CARE
BECAUSE YOU CARE



**MINIMISES
EPIDERMAL
STRIPPING**

**REDUCES PAIN
AT DRESSING
CHANGES**

**PROTECTS THE
PERIWOUND**

**FOR NIL TO LOW
ABSORPTION**

SILICONE FOAM LITE RANGE

WOUND CARE
BECAUSE YOU CARE

FEATURES


- // Breathable // Gentle secure adhesion to the skin // Low friction - water resistant and bacterial barrier //
- // The dressing is easy to remove // Minimal epidermal stripping and pain during removal //
- // The dressing may remain in situ for up to 7 days

ACTIVHEAL® SILICONE ADHESIVE FOAM LITE IS A STERILE WOUND DRESSING THAT IS SOFT, THIN, DISCREET AND CONFORMABLE TO ANATOMICAL CONTOURS.

The dressing is breathable and manages non to low exuding wounds. The dressing is easy to remove and provides a gentle but secure adhesion to skin with minimal epidermal stripping or pain during removal.

ActivHeal® Silicone Foam Lite dressing has a trilaminar structure consisting of a hydrophilic, absorbent, polyurethane central foam layer, and a breathable and low friction, polyurethane film outer layer, which provides a waterproof and bacterial barrier. The wound contact layer is a silicone adhesive coated perforated film.


ActivHeal® Silicone Foam Lite dressing manages exudate in non to low exuding wounds and provides a moist wound environment which aids the wound healing process. The dressing helps to minimise maceration and damage to the surrounding skin. The perforated silicone wound contact layer aids atraumatic removal, enables exudate uptake, and prevents ingress of granulation tissue into the dressing. ActivHeal® Silicone Foam Lite dressing can be repositioned during application or lifted during wear time for observation.

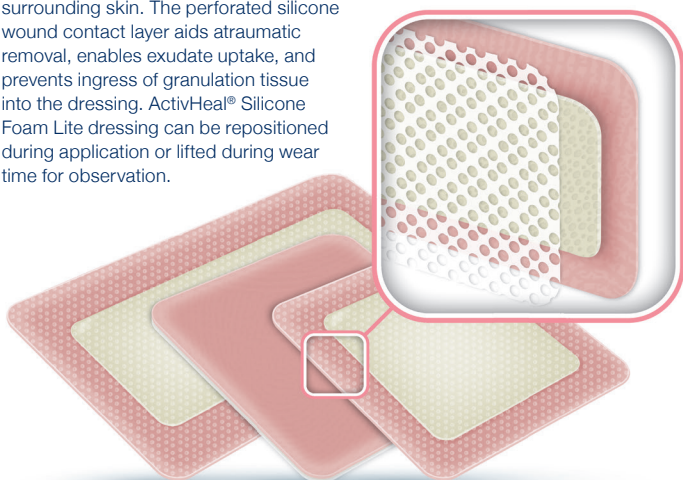

PERFORMANCE

Total Fluid Handling Performance¹

Be confident this foam can handle wound exudate.

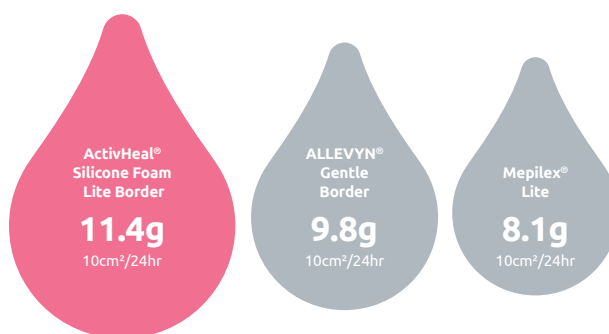

INDICATIONS

ActivHeal® Silicone Foam Lite dressing is indicated for use in the management of non to low exuding, chronic and acute wounds throughout the healing process. The dressing is indicated for use on the following wounds:

- Pressure ulcers
- Diabetic ulcers
- Leg & foot ulcers
- Post-operative surgical wounds
- Trauma wounds (including lacerations, abrasions, skin tears and blisters)
- Superficial and partial thickness burns
- Radiation damaged skin



ActivHeal® Silicone Foam Lite dressings are suitable for use on fragile skin and under compression bandaging.

SIZES AND CODES

ACTIVHEAL®		SILICONE FOAM LITE BORDER			SILICONE FOAM LITE NON-BORDER		
SIZE (CM)	QTY	PRODUCT CODE	DT PIP CODE	NHS SUPPLY CHAIN	PRODUCT CODE	DT PIP CODE	NHS SUPPLY CHAIN
5x5	10	9023687	412-9938	ELA1135			
5.5x12	10	9023694	413-0340	ELA1136			
7.5x7.5	10	9023700	413-0357	ELA1137	9023755	413-0423	ELA1142
10x10	10	9023717	413-0365	ELA1138	9023762	413-0431	ELA1143
15x15	10	9023724	413-0373	ELA1139	9023779	413-0415	ELA1144
10x20	10	9023731	413-0381	ELA1140	9023786	413-0407	ELA1145
10x30	10	9023748	413-0399	ELA1141			

Advanced Medical Solutions Ltd

Premier Park, 33 Road One, Winsford Industrial Estate, Winsford, Cheshire CW7 3RT
 T 08444 125 755 E activheal@admedsol.com W www.activheal.com



References 1. P3103R AMS Data on File

Allevyn Gentle is a registered trademark of Smith & Nephew. Mepilex is a registered trademark of Mölnlycke Health Care.

© 2020 ActivHeal is a trademark of Advanced Medical Solutions // Review date May 2020 // Document code AMS/AH-009-02